

securAcath[®]



**A New
Securement Device
for Drains**

SecurAcath[®] provides improved drainage catheter securement

Reliable securement of drainage catheters is a serious clinical problem. Sutures and adhesives have been used for years with moderate success. Catheter migration and dislodgement are frequent complications of drainage catheters.

How does the SecurAcath work?

- Small, blunt, nitinol securement feet are placed just beneath skin right at the catheter insertion site
- Cover is snapped onto base to affix to catheter shaft
- No sutures or additional skin punctures are needed
- No adhesives needed for securement
- Remains in place for life of catheter
- Works with nephrostomy, general/abscess, biliary, pleural and other drainage catheters



FOLD



INSERT



SNAP





Benefits on Drainage Catheters

Decreased Catheter Dislodgments and Replacement Costs

- Lower cost of patient care
- Minimize therapy interruption

Increased Stability

- SecurAcath secures right at the catheter/skin junction
- Minimizes catheter movement, pistoning and migration

Eliminates Suture Needle Sticks

- No sharps related injuries

Improved Efficiency



- SecurAcath is much faster to use than sutures

360 Degree Site Cleaning While Secured

- Excellent cleaning access around the entire insertion site
- Catheter remains stable and secure during cleaning

Decreased Skin Irritation

- Sutures can become infected, cause skin irritation and breakdown
- Minimize use of skin-damaging adhesives

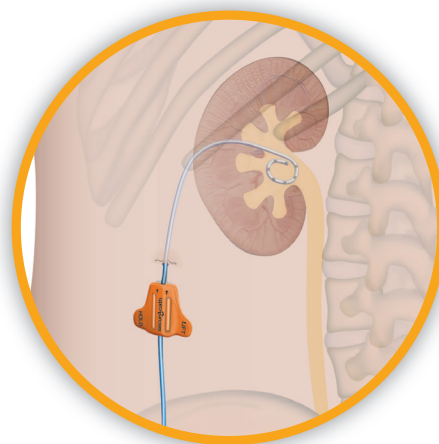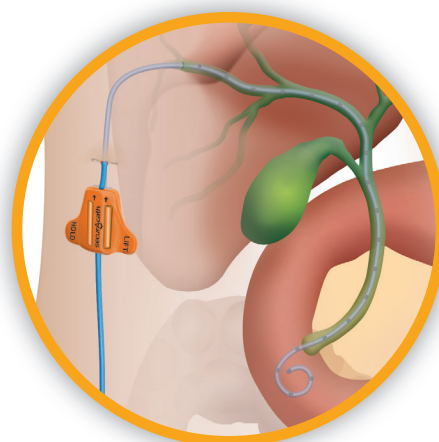

SecurAcath® – For the Life of the Line

Find out more at www.securacath.com

ORDERING INFORMATION



NO.	SIZE	QTY.
400150	6F	Box (10 each)
400120	7F	Box (10 each)
400160	8F	Box (10 each)
400180	10F	Box (10 each)
400200	12F	Box (10 each)

Additional SecurAcath® product information

- Not made with natural latex rubber
- MRI Conditional

More information available at www.securacath.com



TO ORDER:

Call customer service at
866-980-1811
or email order@securacath.com

CAUTION: The SecurAcath is not indicated for use on drainage catheters in the U.S.



181 Cheshire Lane, Suite 100
Plymouth, MN 55441 USA
+1.763.225.6699
www.securacath.com

Please refer to instructions for use for indications, contraindications, hazards, warnings, cautions and directions for use.

CE 0473 www.securacath.com/patents

©2018 by Interrad Medical, Inc. Interrad Medical and SecurAcath are trademarks of Interrad Medical, Inc. All rights reserved.

Download the new
SecurAcath® app!

